



## PORTABLE inLOOP

inLOOP products are made exclusively in the USA. Some electronic parts are not produced in the US and have to be imported.

### SPECIFICATIONS

Covers up to 4' diameter  
22 volt Lithium Ion rechargeable battery.  
Analog audio line input.  
Built in microphone.

### DIMENSIONS

Amplifier 6" x 7" x 1.5"  
Package 10" x 10" x 4"  
5 lbs.

### CONTENTS

Portable inLOOP Projector  
24 volt 2 amp adapter  
Remote Wireless Input  
12 volt 1 amp adapter  
9 volt battery  
Lavalier microphone  
3.5 mm analog cable  
3.5 mm to RCA adapter

### WARRANTY

Portable inLOOP Projector and Remote Wireless Input are warranted for 5 years; excluding damage or failure caused by misuse or neglect

569 South State Street  
Shelby, MI 49455  
info@inLOOP.tv  
(231) 798-2399



**inLOOP**

## Remote Wireless Input



The remote wireless input has a built in microphone. Place it close to a sound source for a clear signal, or use the included lavalier microphone



Any T-coil equipped hearing aid or cochlear device will pick up the signal from inLOOP Portable within a three foot radius. There is a built in microphone for more intimate conversations



## RECHARGEABLE VERSATILE PORTABLE

Connect the remote wireless input device with the built in microphone or with an analog audio cable directly from the TV or other sound source



## PORTABLE inLOOP PROJECTOR



## Use inLOOP Portable upright or laying flat to find the best orientation

**Most tablet computer stands fit inLOOP Portable and provide the perfect listening angle**



Use the built in microphone for conversations with people in close proximity

The built in microphone is activated by depressing the “Microphone/Analog Line” button. This microphone can be active at the same time as the Remote Wireless Input so you can hear people close to you while connected to another device through the Remote Wireless Input.

Insert the included analog cable into the “Input” to use this device to listen to a television or another audio device.

Turn the “Power” switch on to activate inLOOP Portable. It will run for more than 6 hours on the built in rechargeable Li-on battery, or plug it in with the included 24 volt power supply.

Press the “Remote Channel” buttons on inLOOP Portable and the Remote Wireless Input at the same time to synchronize the connection between the devices. Release the buttons when the lights are both lit.

Turn the Remote Wireless Input on with the power switch on the side.

Insert the included analog cable into the “Input” to use this device to listen to a television or another audio device.

The microphone is activated by depressing the “Microphone/Analog Line” button. There is a built in microphone for ease of use, or plug in the included lavalier microphone to use inLOOP Portable as a personal loop system with a speaker, lecturer or teacher.



Use the included 12 volt power supply to conserve the battery in the Remote Wireless Input

